



## LOLER Report

**C1027**

AIG Control Systems Ltd  
Otter Pool Barn  
West Stowford Barton Barn,  
West Stowford,  
Ilfracombe  
Devon, England  
EX34 8PJ

07970 538998

### Visit Details

Visit Type: LOLER  
Order No: 78280/  
Date/Time: 21/06/2024 08:54  
Time Completed: 13:28  
Consultant: Same As Site  
Contact: Simon Brooks

### Job Details

LOLER inspection visit due, June 2024

### Lift Details

Asset /Location: Ground Floor Bedroom  
Make: OpeMed  
Model: Elite 2 Compact  
Serial: 17160734  
S.W.L. 200kg  
Date of  
Manufacture 01/01/2016  
Installed:  
Date of Last  
Examination 21/06/2024  
Next LOLER due 21/06/2025

### Completion Notes

Carry out a Loler inspection of the ceiling track hoist and double gated XY system in the ground floor bedroom-shower room.  
I cleaned the charging and gate points and load tested the hoist.  
I replaced the batteries, as the hoist was raising slower than normal when loaded and when I tested the batteries after load testing the current was too low and they would not hold charge.  
No other defects were found.

### Parts Used

PartID	Description	Qty
1006	12v 2.9Ah Battery	2

### Further Work Required

### VAT Exemption Certificate - Eligibility declaration by an individual

I declare that i am chronically sick or disabled and suffering from \_

I am receiving a qualifying Product or Service/Maintenance visit supplied to me for my domestic or personal use. I claim relief from Value Added Tax under group 12 of schedule 8 of the Value Added Tax Act 1994. The work has been completed.

**Customer Name:** AIG Control Systems Ltd

**Customer Sign:**

**Date:** 21/06/2024

**Rep Name:** Matt Carling

Rep Sign: *Mano*

Date: 21/06/2024



Custom 16

## LOLER Checks

<p><b>Customer</b></p> <p>AIG Control Systems Ltd Otter Pool Barn West Stowford Barton Barn, West Stowford, Ilfracombe Devon, England EX34 8PJ</p>	<p><b>C1027</b></p>	<p><b>Visit Details</b></p> <p>Visit Code: J78280 Order No: TVMC/21/06/2024/Elite 2 Compact Consultant: Same As Site Contact: Simon Brooks Tel:</p>
		<p>Make: OpeMed Model: Elite 2 Compact Serial: 17160734</p>

### Equipment Examined

Question	Answer
EIN number /Asset Number if present:	N/S
What is Manufacture Date?	2017
What is S.W.L. (Kg)?	200kg

### Type of Examination

Question	Answer
Initial Examination	Loler/Service
Examination after Reinstallation	N/A
Examination within a 6 -month Interval (9.3.a.i)	N/A
Examination In accordance with a written scheme	N/A
Exceptional Examination(Please Give Details)	N/A

### Examination Points

Question	Answer
Battery Voltage and Amps (1)- Serial Number of Battery	1.89v - 1.21Ah
Battery Voltage and Amps (2)- Serial Number of Battery	1.51v - 0.56Ah
Check Charger & Docking Points	I cleaned residue from the charging points and dry polished them.
Check Directional Controls/Function	No defects found
Check Emergency/ Controlled Lowering	No defects found
Check Labelling including SWL/ Service Dates	Complete
Are User Instructions Present?	Yes
Weight test/Load Amount (kg)	175kg
Pictures taken of weight test?	No
Limit/ Safety Switches	No defects found
Track Position Related to Lifting Task	Doubled gated XY systems
Track Fixings	Unable to check track fixings as the fixed rails are in the ceiling
Secondary Track	No defects found
Track End Stops	6 of the 8 end stops are fine, but 2 of the end stop rubber bumpers on the fixed rail near the en-suite have been removed. I moved the end stops forward 20mm to prevent the traverse rail from twisting when

	put onto charge.
Gate system Electrical/Mechanical	No defects found
Hoist Rollers	I lubricated the rollers on the bathroom XY system as they were grumbling. Traverse rail runs smoother now.
Motor Carriage	No defects found
Turntable	N/A
Safety Bar and Covers	No defects found
Safety Bar Roll Pin Checked	N/A
Retaining Pins	N/A
Split Pins in Carrier Trolley (Robin CTH)	N/A
Track Joins/ Interlocks including Gates	No defects found
Check lifting tape for signs of wear	Lifting tape has a rolled edge 6 inches long mid way down. Please monitor and report it if the tape starts to fray.
Abnormal Noise or Resistance	None
Dangerous Defects Found?	None
If Yes Please detail items which are safety critical now	None
Potentially dangerous defects (not safety critical) Found?	None
If Yes Please detail for monitoring	Monitor the condition of the lifting tape.

### Consent Request

Question	Answer
Does client give consent for pictures to be used on social media?	No

**Rep Name:** Matt Carling

**Rep Sign:** 

**Date:** 21/06/2024